Nexcare

Steri-Strip™ First Aid Skin Closures


en

Steri-Strip® Skin Closures are adhesive strips that are secure, close and support small skin cuts and wounds. To be used as directed by physician or health care provider for treatment of small skin wounds in conjunction with skin sanitors and disinfectants or after their removal, for wound support. Do not use on deep or jagged cuts, puncture wounds, lacerations or on hot, burnable, breakable.

Directions for use:

- 1 Clean and dry skin. Remove excess hair.
- 2 Fold one end of the liner and tear off the strips separately.
- 3 Smooth together the skin edges with fingers. Apply strip without tension using gentle, firm pressure. Smooth from the center outward.
- 4 Additional closures may be applied parallel to the wound and overlapping approximately 12 mm from the closure ends to improve adhesion.

To remove gently peel skin closure down and slow, towards the center of the skin, while supporting the skin.

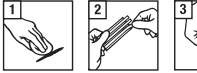Contraindications:

- Do not use where good adhesion cannot be obtained due to the presence of wound fluid, skin oils, moisture, or hair.
- Do not use on puncture, deep, or infected wounds. Consult your health care provider for signs of infection or if your signs of infection (fever, pain, swelling, itching, or burning).
- Do not use on high tension wounds which cannot be easily held together with fingers.
- Do not use on a securement device, such as on holding band in place.

Warnings: To reduce the risk of infection, do not use tearsips.

Precaution:

- As with all adhesive products applied to the skin, a small percentage of individuals may experience skin irritation, discoloration, or injury during wear or upon removal. • Resin causes a post-operative sensitivity, body joint movement or applying skin closure, under tension may cause skin injury, including blistering or loss of adhesion. • Use with additional adhesive products may cause blisters or skin injury. • If the wound is deep and bleeding continuously seek medical assistance immediately. • This product and its removal is not intended for use except in the home.

Please report a serious incident occurring in relation to this device to 3M and the local competent authority.

Always read the label. Do not use. If discomfort or pain results, persist or increase, discontinue use and consult a physician.

© 2012 3M Corp. 2012-06

Storage/ Shelf life/ Disposal

Store at room temperature. Avoid heat and humidity. Do not use if expired or if expiration date is missing or illegible. Dispose of after final guidelines.

3M Deutschland GmbH
Health Care Business
Carl-Schurz-Str. 11
41453 Nettesheim, Germany

For more information, see:
HCS@Germany.3m.com
3M and Nexcare are trademarks of 3M.
© 2022, 3M. All rights reserved.
Date of issue 2022-06

Made in USA with globally sourced materials

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